

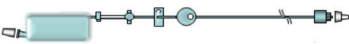






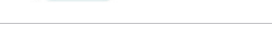

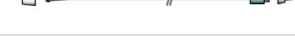
Rythmic™ Perf +

ORDER
CATALOGUE

References	KP5.04.188.2 Rythmic™ Perf + KP5.04.274.2 Mini Rythmic™ Perf+ KP5.04.287.1 Rythmic™ Perf+ w. permanent key
Delivery modes	- Continuous - Volume/time or Volume/rate - Auto ramp - Intermittent - 25 steps
Flow rate and Accuracy	0.10 to 400ml/h @+/- 5% (2 decimal places in V/T and Auto ramp modes)
Programming Units Concentration	ml/mg/µg 0.1 to 99.9 mg/ml and 0.1 to 999.9 µg/ml
Programmable infusion time	1 min to 199h59
Auto ramp, up and down time	0 min to 9h59
Volume to infuse	1 to 9999 ml
Special functions	- Early taper down - Delayed start (up to 24 hours) - Mandatory priming (enabled, warning only, disabled) - Automatic priming with the pump - Lockable keyboard
Therapy monitoring	Ready to connect to www.micrelcare.net for therapy monitoring through the web. History Plug & Play printout and downloads on PC or via Micrelcare™
KVO	0.1 to 20.0 ml/h
Automatic restart after occlusion release	Yes
Occlusion limits	3 levels: low= 7psi (0.5 bar), medium=13psi (0.9bar), high=18.8psi (1.3 bar)
Safety features	Air in line eliminator, free flow protection, lockable protective covers for drug container
Alarms & warnings	Upstream and Downstream Occlusion, near end (in time or volume) and end of infusion, low and depleted battery, open mechanism, KVO, incorrect programmed rate, preventive maintenance warning
Power supply and autonomy	9V alkaline: 7 days 2ml/hr
Preventive maintenance	Every 2 years
Weight with battery	KP5.04.188.2 Rythmic™ Perf + : 335g KP5.04.274.2 Mini Rythmic™ Perf+ : 300g KP5.04.287.1 Rythmic™ Perf+ w. permanent key : 335g
Dimensions	KP5.04.188.2 Rythmic™ Perf + : 130 mm x 133 mm x 46 mm KP5.04.274.2 Mini Rythmic™ Perf+ : 130 mm x 75 mm x 46 mm KP5.04.287.1 Rythmic™ Perf+ w. permanent key : 130mm x 133 mm x 46 mm



DEHP free set range

Rythmic™ Administration Set range for Rythmic™ Perf + Systems			
Mini Full Set 100 mL KE1.EE.177.9		Spike Set S KM1.EE.155.9	
Full Set 200 mL KE1.EE.185.9		Spike Set S Y KM1.EE.187.9	
Y Mini Full Set 100 mL KE1.EE.195.9		Short Set 100 mL KE1.EE.158.9	
Y Full Set 200 ml KE1.EE.196.9		Valve Extension Set KM1.EE.141.8	
Spike Set KM1.EE.099.1		Valve Extension Set with filter 0,2 um KE1.EE.220.1	

Accessories For Rythmic™ Perf+ Systems

	Carrying bag for Rythmic™ KM1.YY.415.2		Carrying bag 500 ml for Rythmic™ and IP Connect KM1.YY.318.2		Mini Rythmic™ Bag 500 ml KM1.YY.365.1
	Mini Rythmic™ Bag KM1.YY.400.1		Disposable Carrying bag for Mini Rythmic™ KM1.YY.566.1		Banana carrying bag 200 ml for Mini Rythmic™ KM1.YY.506.2
	IP Connect Pack nc KP5.04.259.2		Micrel Power Adaptor nc (Available also UK and USA Nema) KS5.01.615.4		Pole Clamp mini Rythmic™ KS5.04.017.5
	Rythmic™ holder with Pole Clamp KS5.01.430.3		Carrying bag for Mini Rythmic™ 500 ml KM1.YY.563.1		

96155 Rev:1 Release date 2019-11-7